



SimuEYE BIO Shield™



Protective Shield For Your Binocular Indirect Ophthalmoscope

PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE USE

PARTS INCLUDED



Bracket System includes:

- 1 x Scope Mount (with ridges)
- 1 x Shield Mount
- 1 x Wall Mount (smooth edges)



1 Shield Piece covered by protective film



1 Wall Mount Plate



Adapter Brackets various sizes



Extra Adhesives save for later use



1
Unpack Bracket System and separate magnets by sliding apart.



2
Clean the surface of the indirect with an alcohol wipe.



3
Remove adhesive protectors from Scope Mount.



4
Attach Scope Mount to the indirect at mid-line. Keep logo straight. Press firmly.



5
Line up the logo of the Shield Mount to the Scope Mount.




6
Allow the Shield Mount magnet to attach to the Scope Mount.

For assistance assembling or using this product please contact our customer service department Mon - Fri 8am - 4pm PT
1-818-699-6363 or e-mail us at info@simuleye.com

www.SimuEYE.com



Remove adhesive protector from Shield Mount.

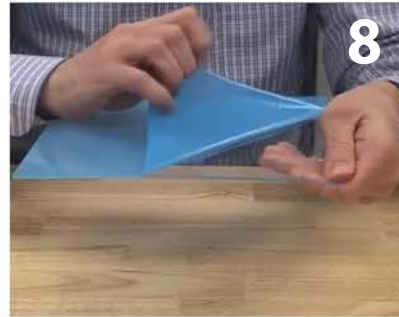

Remove protective coating from both sides of the Shield.



Line the Shield up to the adhesive and press firmly.



Remove adhesive protector from Wall Mount and place over the Shield Mount on back of Shield.



For best results, let the adhesives cure for up to 12 hours before use.



Remove the adhesive protector from the back of the Wall Mount Plate and press firmly to wall.

ADAPTER BRACKET INSTALLATION

The Adapter Bracket is only required for the Heine Omega 150, Keeler Fison or other models with rounded architecture where the Shield attaches.

To install the Adapter Bracket, complete these steps BEFORE steps 3-12 above.

1. Compare the curvature of the Adapter Brackets to your Indirect and choose the best fit.
2. Peel away the adhesive protector from the Adapter Bracket and firmly press onto the Indirect at mid-line.
3. Complete steps 3-12 above, attaching the Scope Mount to the Adapter Bracket.

REMOVAL & CLEANING INSTRUCTIONS

To remove the Shield, simply pull the bottom edge up and away from the scope.

Use the same motion to remove the Shield from the Wall Mount Plate.

Wash away any debris with warm water and soap.

Use a lint-free cloth with alcohol or a disinfecting wipe to clean the Shield between uses.

DO NOT use abrasive cleaning products or brushes as they can leave scratches.

WARNING! STRONG MAGNETS

The SimuEYE BIO Shield™ is not FDA cleared and has been authorized by FDA under an emergency use authorization (EUA) for use by healthcare providers as personal protective equipment for the duration of the circumstances justifying the EUA declaration under Section 564(b)(1) of the Act, 21 USC 360bbb-3(b)(1) unless the EUA is terminated or revoked sooner.