

# Instructions for Use: LED Three Character Test (15305)

Contents: LED Three Dot Test, 2 D Cell Batteries, Red & Green Spectacles (15105)

The LED Three Character tests patients not responsive to the Worth Four Dot Test, patients who do not recognize colors or have low color perception for diplopia and eye fusing issues.

For an electronic version of the instructions visit <https://guldenophthalmics.com/resources/>

To insert the batteries, turn the top collar counterclockwise while firmly holding the base. Place 2 D batteries (included) in with the + direction facing up. Replace the collar by turning clockwise to secure it to the base. To conserve the batteries and prevent corrosion, remove them when not in use.

Hold the light three feet from the patient wearing the red and green spectacles. The red lens should be on the right eye and the green on the left eye. If the test is not performed at three feet distance proper diagnosis may not be obtained.

If suppression occurs, only two characters will be seen. With the red lens on the right eye, if the right eye is suppressed the girl will disappear. If the left eye is suppressed, the elephant will disappear.

If a singular ball, girl, and elephant are seen – both eyes are being used and the patient has at least grade one binocular vision.

This device is reusable.

EDA 1158

10/15/2021

Any serious incident that has occurred in relation to this device should be reported to Gulden Ophthalmics and the competent authority of the Member State in which the user and/or patient is established in the EU.



Gulden Ophthalmics  
225 Cadwalader Avenue  
Elkins Park, PA 19027



EC REP

Qarad EC-REP BV  
Pas 257  
2440 Geel  
Belgium