

Luedde Exophthalmometer (16104)

Instructions for Use:

Purpose:

Exophthalmometers are utilized to measure the exophthalmos of patients' eyes.

Warning/Precaution:

Do **Not** use alcohol to clean.

The Luedde can be cleaned with mild soap and water.

Operation:

Place the curved end of the Luedde to fit the apex of the deepest angle of the lateral orbital notch. Have patient gaze at a fixation target straight ahead of them. Move to the patient's side and look through the Luedde at a 90 degree angle to their gaze. Line up the scales with each other to straighten Luedde. Use the scale to measure the corneal apex in millimeters.



225 Cadwalader Avenue, Elkins Park, PA 19027

(800) 659-2250 Fax (215) 884-0418 e-mail: info@guldenophthalmics.com

www.guldenophthalmics.com

Rev. EDA 1108 || 6/18/2021