

# I-Borg Instructions

Gulden Ophthalmics, Inc. • 225 Cadwalader Avenue • Elkins Park, PA 19027 • USA  
www.GuldenOphthalmics.com • 800-659-2250

This product is designed to be used in conjunction with the Phake-i product (item # 17004).

Please note that this is a tool for **TEACHING OR PRACTICE ONLY**, and it should **NEVER UNDER ANY CIRCUMSTANCES BE USED ON LIVING PATIENTS**.

<p><b>Setup:</b></p> <p><i>Inserting the Phake-i</i></p> <ol style="list-style-type: none"><li>1. Remove Phake-i from cylinder container.</li><li>2. Turn the I-Borg over and pull back on the spring-loaded black knob on the connector bar on the back of the head. This will lower the platform.</li><li>3. Place the Phake-i, base first, onto the lowered platform.</li><li>4. Slowly raise the platform back up so that the Phake-i sticks out of the orbital socket on the head. The Phake-i should be able to rest on the platform and stay put.</li></ol>	<p><b>Setup:</b></p> <p><i>Connecting the fluid collection bag</i></p> <ol style="list-style-type: none"><li>1. The fluid collection bag is used during irrigation so that the fluids will run down into the basin and go through the drainage hole at the top of the head collecting in the bag via the vinyl hose.</li><li>2. Insert drainage tube into the fluid collection bag by connecting the blue end of the hose to the white cap on the bag. Spin to lock.</li><li>3. Connect the fluid collection bag to the I-Borg head by inserting the white end of the hose into the blue connector. Spin to lock.</li></ol>
--	---

Notes:

- The device should be cleaned after use with a mild solution of antibacterial soap and water.