



Instructions for Use: Contact Lens Polisher Kit (15172)

1. Install the flat foam insert into polisher head.
 2. Put polish on the foam insert.
 3. Place the contact lens convex side up on the tip of your index finger.
 4. Hold the polisher in your other hand and place the polisher head over the contact lens on your finger.
 5. Turn on the polisher by sliding the switch up.
 6. Control the speed by increasing or decreasing the pressure on your finger. The harder you push the slower the motor spins and the more aggressive the polishing becomes.
-

Change inserts to polish the concave side.

1. Install the cone foam insert into polisher head.
 2. Put polish on the foam insert.
 3. Place the contact lens concave side up on the tip of your index finger.
- Repeat steps 4 to 6.
-

Battery Replacement:

Slide cover on the bottom to open the housing replace both AA batteries.